Vectra® Lite Fixed Walker
Vectra Basic Fixed Walker
Vectra Premium Fixed Walker

Breg, Inc.
2885 Loker Ave. East
Carlsbad, CA 92010 U.S.A.
P: 800-321-0607
F: 800-329-2734
AW-1.90271 REV C 0812

E/U authorized representative
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany

⚠

W

A

R

N

I

N

G

WARNING: CAREFULLY READ USE/CARE INSTRUCTIONS AND WARNINGS PRIOR TO USE.

WARNING: DO NOT REMOVE WALKER BOOT UNLESS INSTRUCTED BY YOUR MEDICAL TREATMENT PROFESSIONAL.

WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.

WARNING: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

WARNING: TAKE APPROPRIATE PRECAUTIONS WHEN WALKING ON SLIPPERY SURFACES, UNEVEN TERRAIN, OR ANY OTHER PROPERTY CONDITION WHERE THE EXERCISE OF CAUTION APPEARS NECESSARY.

WARNING: DO NOT OPERATE A MOTOR VEHICLE OR HUMAN POWERED VEHICLE WHILE WEARING THIS WALKER BOOT.

CAUTION: FOR SINGLE PATIENT USE ONLY. OUTER SOLE MAY DETERIORATE WITH MULTIPLE USES.

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- Vectra Lite, Basic and Premium Fixed Walker**
- A Apply liner to leg.** With patient seated, leg(s) bent and feet off floor, position tongue pad as needed for maximum comfort, close liner snugly.
- B Slide walker onto leg over the liner straight up directly under foot.**
- C Fasten straps.** Start with straps 1 and 2 closest to the toes (Straps may be trimmed to desired length).
- D Position walker with uprights centered over the malleoli (ankle bones).** Ensure heel is positioned at the rear of the Walker.
- E Remove quick-pulls from the uprights and discard.** Attach uprights to the liner. Press firmly.
- F Fasten straps.** Affix straps 3 through 5 to the outside of the uprights by alternating the buckle placement. Then fasten straps 3 through 5.

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
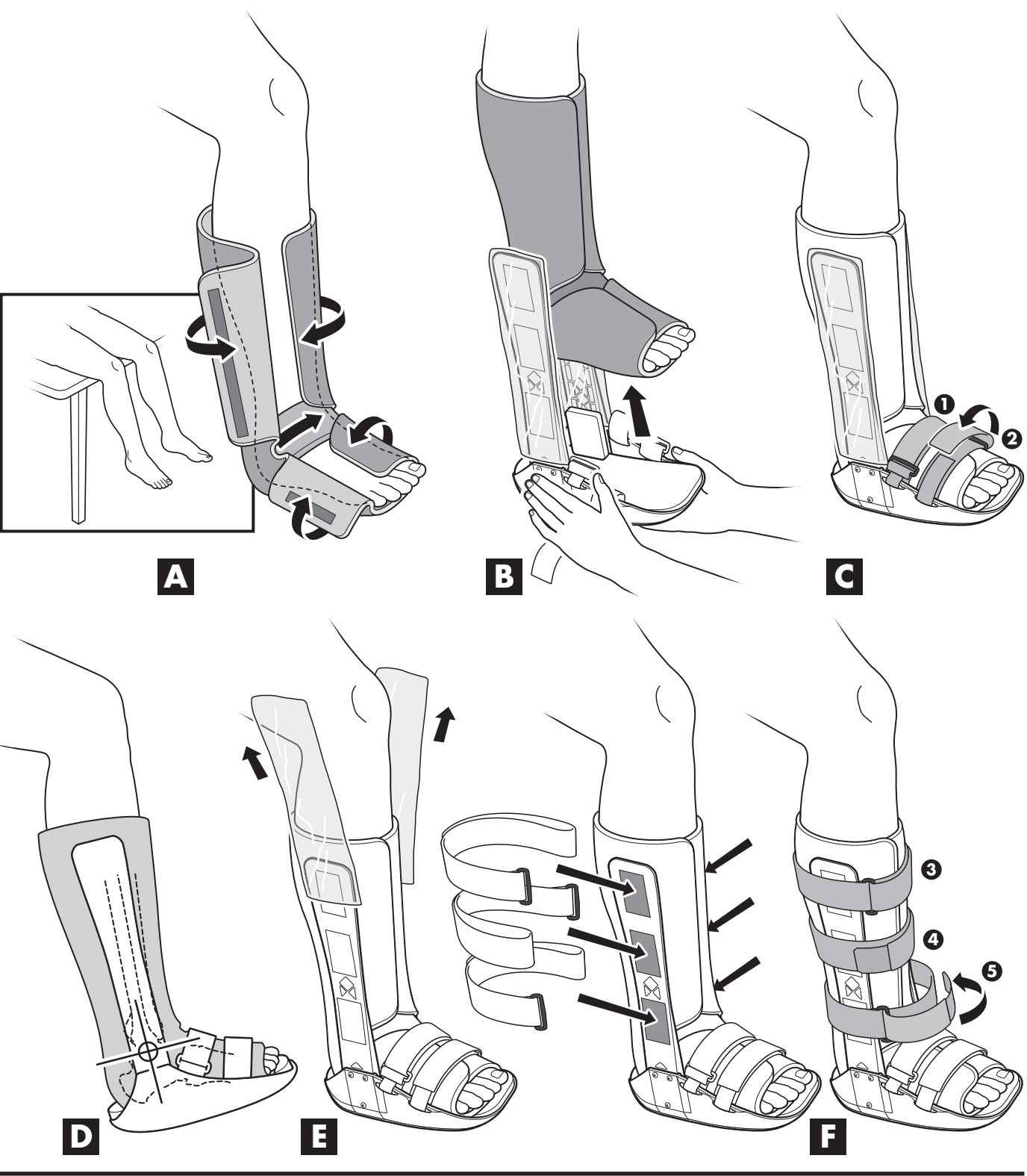
Optional: For maximum liner fixation and patient comfort, attach the 1 x 1/2" hook pieces to the liner at malleoli foam pads (or on the foot bed of the walker).

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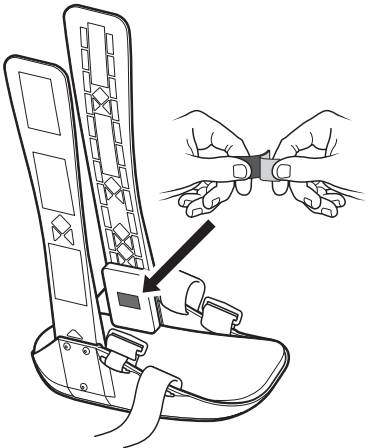Vectra Premium Fixed Walker

- A Apply liner to leg.** With patient seated, leg(s) bent and feet off floor, position tongue pad as needed for maximum comfort, close liner snugly.
- B Slide walker onto leg over the liner straight up directly under foot.**
- C Fasten straps.** Start with straps 1 and 2 closest to the toes (Straps may be trimmed to desired length by removing tab at end of straps, trimming and reattaching tab.
- D Position walker with uprights centered over the malleoli (ankle bones).** Ensure heel is positioned at the rear of the Walker.
- E Remove quick-pulls from the uprights and discard.** Attach uprights to the liner. Press firmly.
- F Fasten straps.** Fasten remaining straps 3 through 5.

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
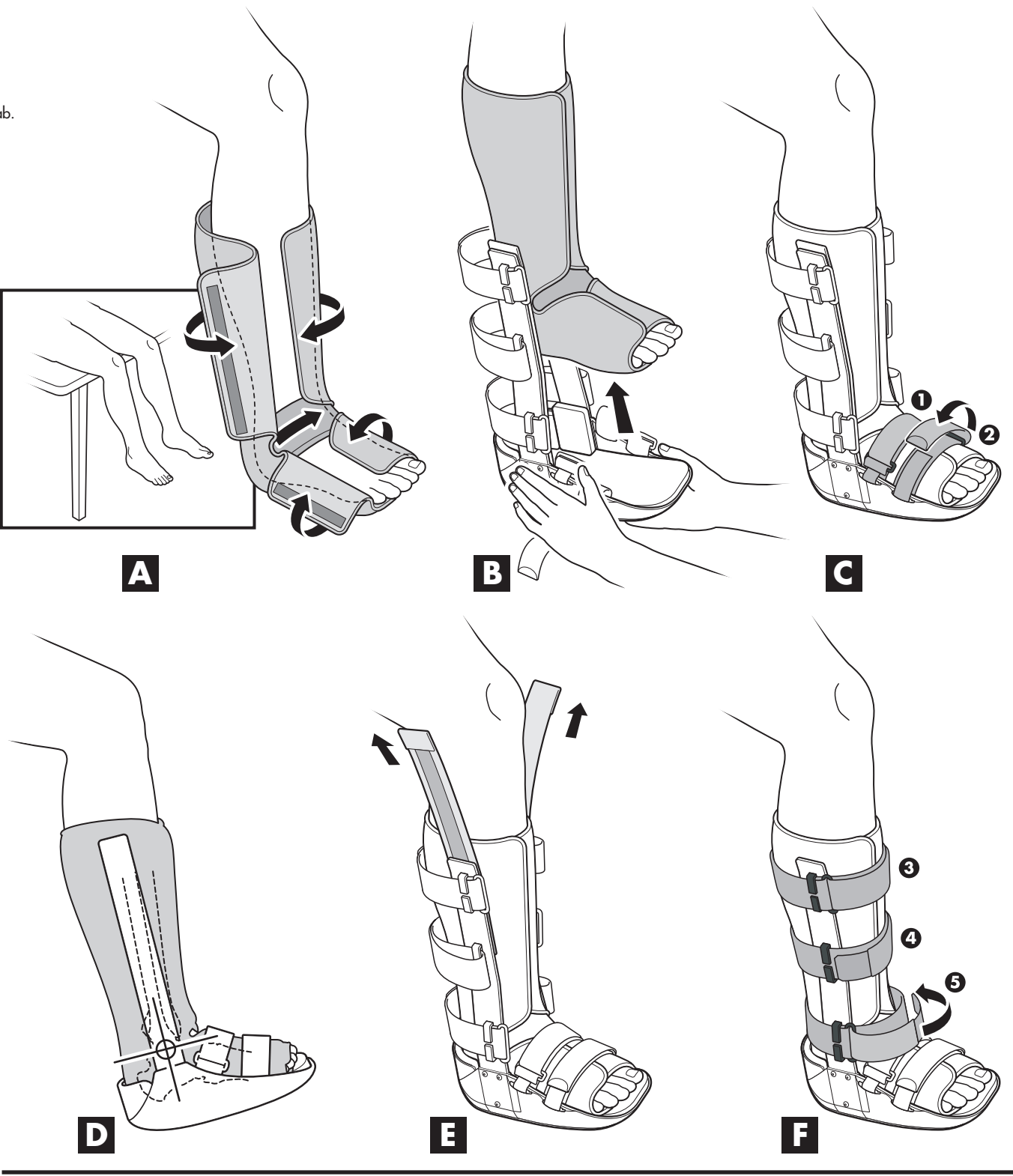
Optional: For maximum liner fixation and patient comfort, attach the 1 x 1 /2" hook pieces to the liner at malleoli foam pads (or on the foot bed of the walker).

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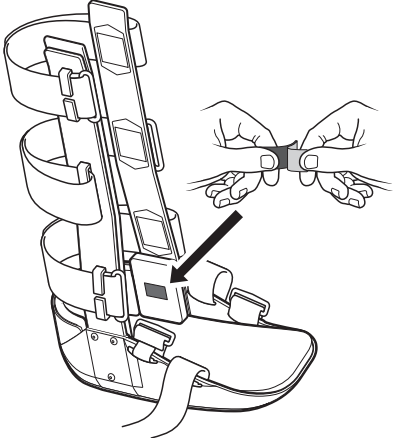Optional heel clip removal Pull up on the heel clip.

Optional heel clip replacement Lift up foot bed padding and foot plate. Slide heel clip into place using the dove-tail slot guides. Release the foot plate and padding and push down into proper position.

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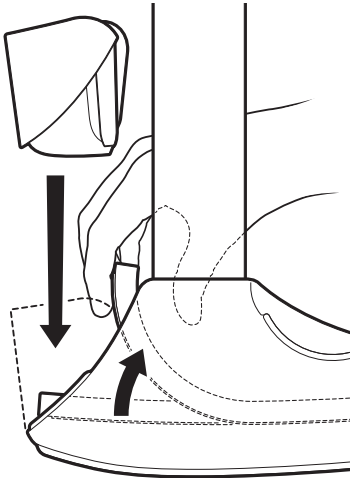
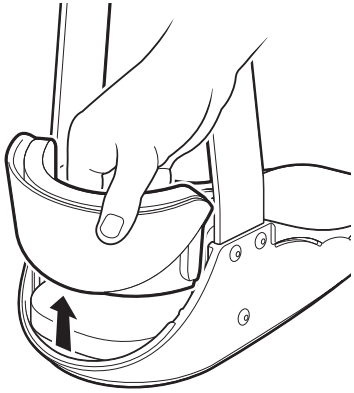
USE AND CARE OF YOUR VECTRA:

- After initial application, the VECTRA may be removed and reapplied without removing liner from boot.
- Hand wash the liner and straps with mild soap, air dry. Do not place liner or straps into a mechanical dryer.
- Extra liners with strap sets are available from Customer Care: (800) 321-0607.

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